

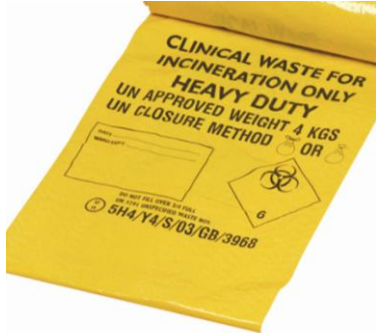





THREE STEP SPILL CONTROL

ABSORB	DISINFECT	REMOVE
		
<p>Put on gloves, apron and mask.</p> <p>Apply Andarta Absorbent Powder uniformly over spillage until completely covered.</p> <p>When liquid has congealed, dispose of using scoop and scraper and place in clinical waste bag.</p>	<p>Use absorbent pad and wipes with Andarta Cleaner and Sanitiser spray to cleanse the affected area.</p>	<p>Place the used scoop, scraper, gloves and apron into yellow waste bag and tie securely.</p> <p>Dispose of the contaminated materials in an approved way.</p> <p>IMPORTANT - DO NOT RE-USE ANY ITEMS.</p>
<p>SPILL KIT REFILLS INCLUDE:</p> <ul style="list-style-type: none"> 1 x Disposable Gloves 1 x Disposable Apron 1 x Disposable Odour Mask 1 x Absorbent Pad 1 x Scoop & Spatula 6 x Wipes 1 x Waste Bag & Ties 		

Keep out of the reach of children

Email: info@andarta-sales.com

Reorder code: 46-004